


User Manual Step-Hear Application

Functions-


	Function	Page
1	More Devices	1
2	Setting	1-3
3	Hot Key for an Internet Page	3
4	Message #2	3
5	Message #4	3
6	What is Around me?	4
7	Message #3	3
8	Message #1	3
9	Hot Key for a Phone Call	4
10	Name of Current Device	4

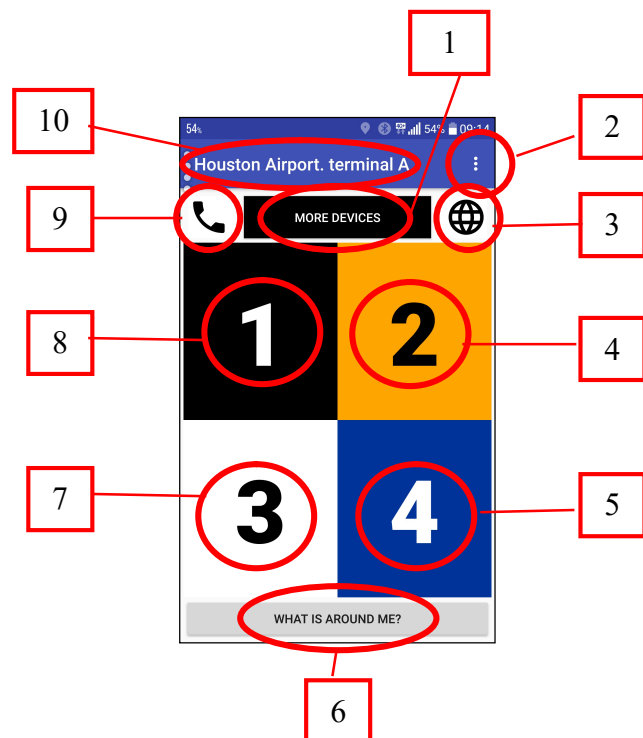

List of Step-Hear Devices

Devices Connected by Bluetooth, can be choose one to operate


Devices **Not** Connected by Bluetooth with estimated distance from the user




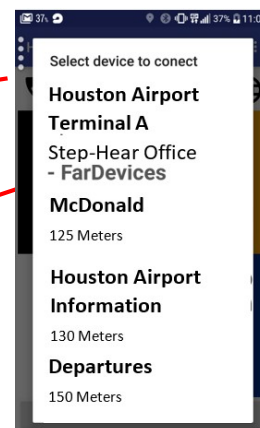


Setting / Step-Hear Audio Sign


Setting / Call-Hear 111

Show device list- Technician

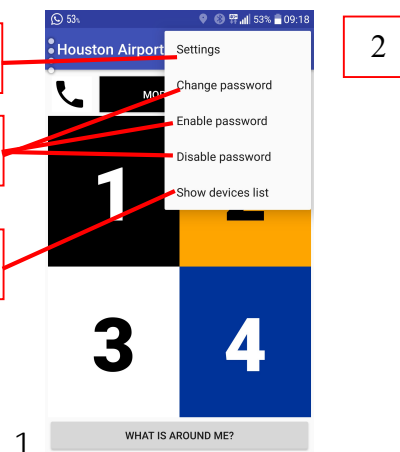

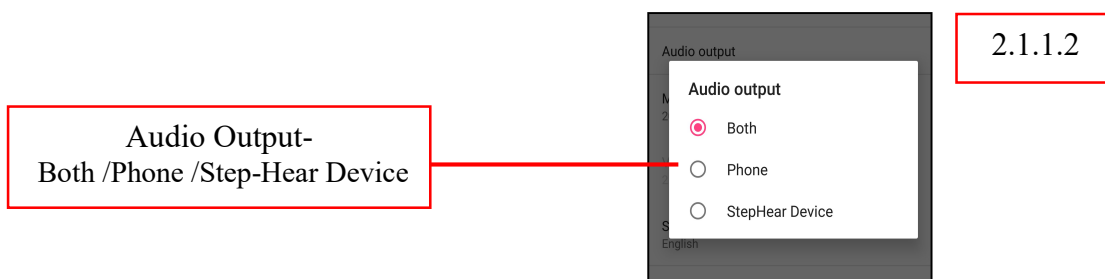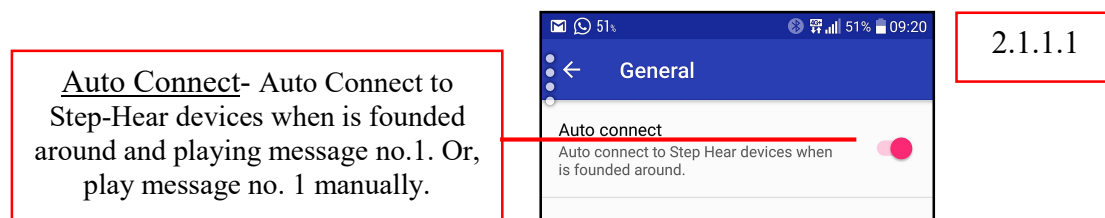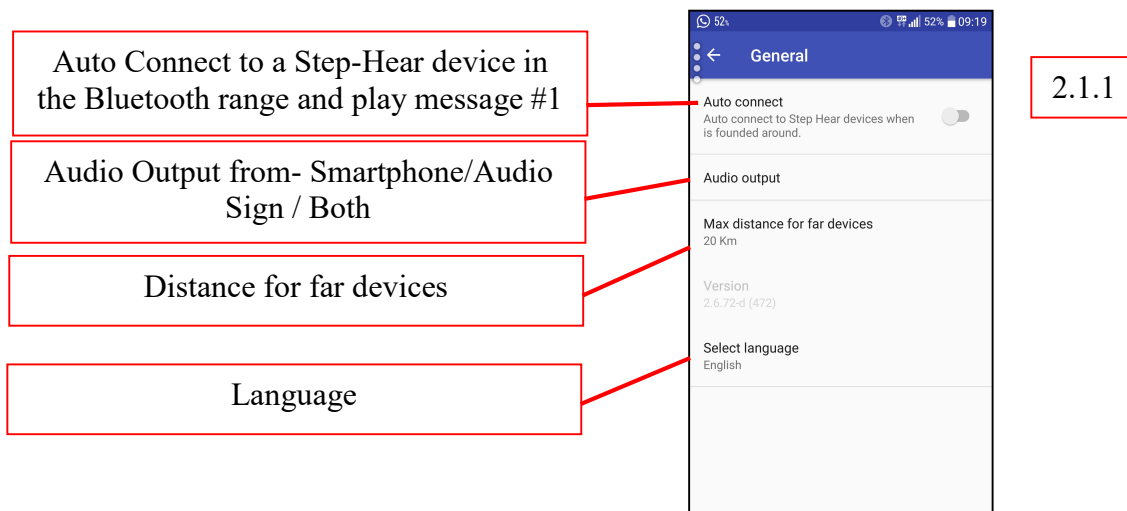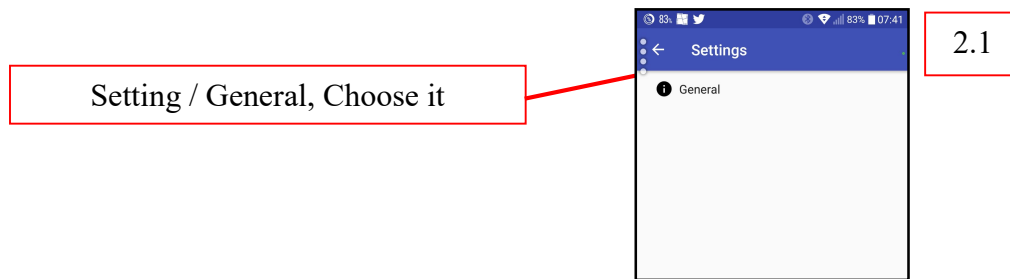


Set the Maximum distance
for devices show on –
“Devices **Not** Connected”
(1)

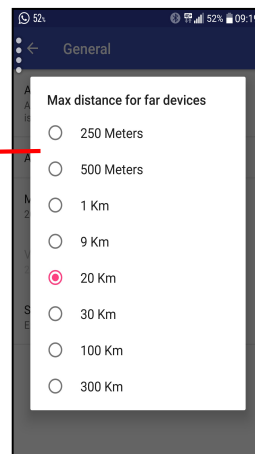

2.1.1.3

Setting Language

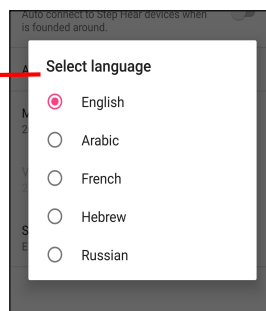


2.1.1.3

Connecting to the Web Page, provided
by the Site



3

Playing Audio Message from box
1;2;3 (No message at 4)
By touching every button for 1
sec, it announces the title of the
message at this device (Entrance,
Elevator etc.)

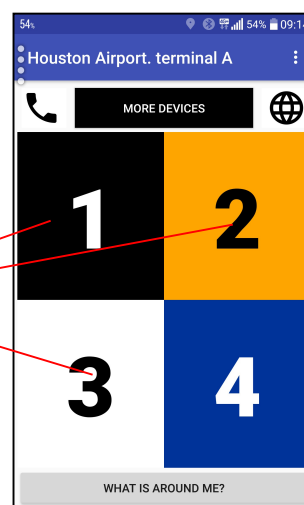

4

